


CODE:RA0001

PRODUCT

Breathing Circuit

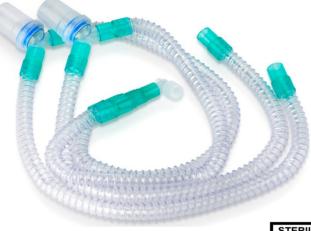

STERILE

MATERIAL

Polyethylene (PE)

MODEL

Corrugated

SPECIALIZATION

Adult, pediatric & neonatal

SPECS

- § Low airflow resistance directly bonded connectors to the tube for avoiding leakage risks.
- § Bio-compatible and Phthalates free material reduces allergy risk in patients.
- § Standard 22mm/15mm/10mm connection meet 1S0-5356-1.
- § Sterilized by ethylene oxide, and the residual amount of ethylene oxide = $<10\mu g/g$.

APPLICATION

For ventilator & anesthesia machines

CONTRAINDICATION

No contraindications.

PRECAUTION

- § Single use only, it is strictly forbidden to use it if the package is damaged and discard it after use.
- § The production batch is on the seal or packaging lab -el, and the sterilization date is shown on the outer box.
- § This product is sterilized by ethylene oxide.

INSTRUCTION

- **1**. Select the applicable model and confirm that the ou -ter packaging is intact before use.
- 2. Connect the circuit to anesthesia machine.
- **3**. After use, discard it as medical waste to avoid pollu-tion.

STORAGE

- § This product should be loaded and transpor -ted in covered carriages and cabins. Keep it clean, avoid heavy pressure, direct sunlight and water.
- § This product should be stored in non-corrosive gas, well ventilated and clean environment. Keep away from fire sources. Relative humidity no exceeded at 80%.





Direccion: R04, 5TH floor,Xinhua Insurance Building,171 Mintian Road, Futian District,Shenzhen City,China,518000 **Tel**: 086-0755 23006645 (10:00AM-5:00PM UTC+8)

Email: info@Medicalets.com